

Real-T Endo

ModuPRO® Artificial Endo Module AE07-12 and Placement of Real-T Endo™ #8

Real-T Endo #8 Tooth
(Item # RT_AE401_08 or RT_AE401_08RL).



Maxillary anterior endo module.
(Item # AE07-12).




Place Real-T Endo #8 Tooth in the AE7-12 module until aligned with the occlusal plane of adjacent teeth.



Gently tighten the retaining screw on the lingual side with a small Phillips screwdriver until the Real-T Endo #8 Tooth is secure. DO NOT OVER TIGHTEN AS YOU WILL DAMAGE THE TOOTH AND/OR MODULE.



Place the AE7-12 T module on the maxillary Carrier Tray.

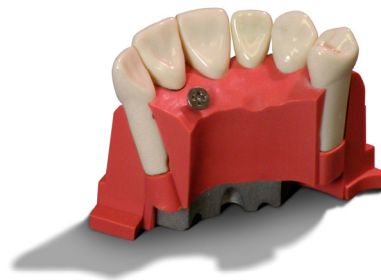

ACADENTAL

www.acadental.com