

Real-T Endo™ Series

ModuPRO® Artificial Endo Module AE13-15 and Real-T Endo™ #14 in Breakaway Insert

Acadental is proud to announce a revolutionary and innovative product for endodontic education. You can now perform an endodontic procedure on a multi-rooted artificial endo tooth in a manikin and remove it afterwards. This allows the user to visualize complete treatment of the canal(s) as well as check for perforations that may have been made through the canal or pulp floor during the procedure.

Below are illustrations on how to place the Real-T Endo #14 tooth with Breakaway Insert™ into the AE13-15 module and remove the tooth after the endodontic procedure.



Fig. 1
Real-T Endo #14 and tooth #13 encased in Breakaway Insert



Fig. 2
AE13-15 Artificial Endo Module with #15 tooth



Fig. 3
AE13-15 Artificial Endo Module Occlusal View



Fig. 4
Gently push the Breakaway Insert into the opening in AE13-15



Fig. 5
Illustration of Breakaway Insert completely inserted into AE13-15 Module. Place this module onto the upper Carrier Tray with modules 2-6 and 7-12 for a complete maxillary arch.



Fig. 6
To remove the Breakaway Insert, simply push the Insert out with a small object with a round end such as an instrument handle.



Fig. 7
To completely remove the Real-T Endo #14 tooth located inside the Insert, simply twist open the Insert with a small coin or similar object and remove the tooth.

