

# Real-T Endo™ Series

## Real-T Endo #8 Tooth Instructions

Receiving excellent reviews from endodontists throughout the world due to the realistic anatomy and life-like canal size, the Real-T Endo #8 tooth allows you to practice and train with multiple endo procedures.

Below are illustrations on how to place the Real-T Endo #8 in the AE7-12 T Artificial Endo module.



**Fig. 1**  
Real-T Endo #8 Tooth



**Fig. 2**  
AE7-12 T Artificial Endo module with teeth #7, #9, #10, #11 and #12



**Fig. 3**  
Place the Real-T Endo #8 Tooth in the AE7-12 module until aligned with the occlusal plane of the adjacent teeth



**Fig. 4**  
Gently tighten the retaining screw on the lingual side with a small Phillips screwdriver until the Real-T Endo #8 Tooth is securely tightened, but no more, as this may damage the tooth.



**Fig. 5**  
Place the AE7-12 T module on the maxillary Carrier Tray and retain with any posterior ModuPRO modules.