


ModuPRO Overview

Thank you for choosing the ModuPRO, which is designed to universally fit most standard posts and simulation systems worldwide. When mounting in a simulation system, remove the four screws attaching the articulator to the upper and lower Carrier Trays.

As a reminder, after use, be sure to wipe dry the magnets and surrounding area on the Carrier Tray to mitigate corrosion.

ModuPRO Calibration and Teeth Replacement

Acadental calibrates the ModuPRO prior to shipment, but it may change after shipping or repeated use. The adjustable articulator, designed and manufactured by Acadental, provides an easy and quick manner in which to calibrate your model allowing lateral and protrusive movement. Simply loosen both of the Shoulder Bolts on the adjustable articulator. Hold the maxillary and mandibular Carrier Trays together until the ideal occlusion is achieved. Tighten both of the Shoulder Bolts to lock the occlusion in place. (These steps may need to be repeated.)

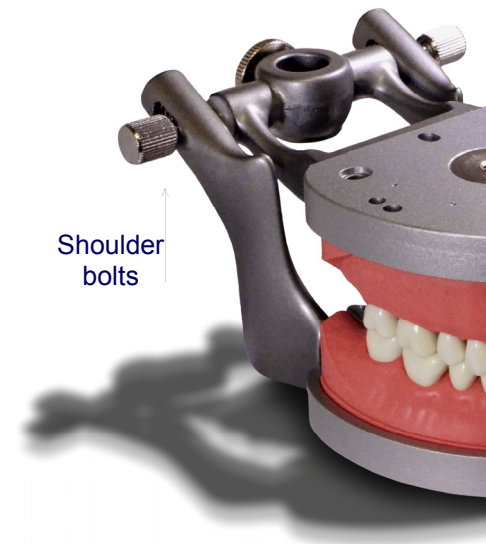

When replacing teeth, tightly push into the correct socket with small indentation facing the lingual side and fasten with Acadental's 1.56mm screws until tight to the turn.

Do not attempt to interchange other manufacturers' teeth with Acadental's replacement teeth.



ACADENTAL