


Attaching the **ModuPRO**[®] to a Darwin Manikin

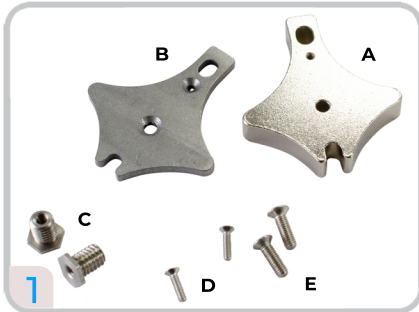

Darwin Manikin Adapter set (AM_DMADAP)
A: Upper Adapter Plate B: Lower Adapter Plate
C: M6 Screws D: M2 Screws E: M3 Screws



Thread an M6 screw in the center of the upper and lower Carrier Trays.



Remove all four screws that are fastened to the Carrier Trays and detach the articulator.



Place the Oral Cavity Cover around the Carrier Trays and position the bite correctly.



Line up the articulator with the holes on the back of the Oral Cavity Cover and Carrier Trays. Fasten all four screws back into place.




Using the screw driver, poke a small hole in the top and bottom of the Oral Cavity Cover at the indentation.



Line up the Upper Adapter Plate on the upper Carrier Tray. Fasten an M3 Screw through the center, and then an M2 Screw distally. Make sure the Adapter Plate has the countersunk side facing up.



Line up the Lower Adapter Plate on the lower Carrier Tray. Fasten an M3 Screw through the center, and then an M2 Screw distally. Make sure the Adapter Plate has the countersunk side facing up.

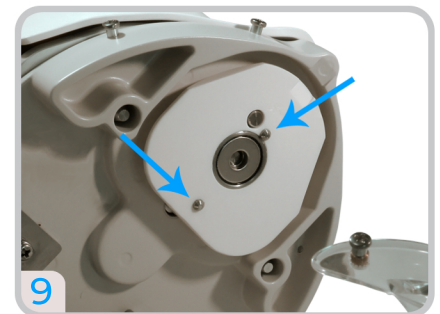

The Adapter Plates will line up with the pins in the upper jaw and magnetically attach to both the upper and lower jaws.



The ModuPRO is now fully attached to the manikin.

***Note: If attaching without the Oral Cavity Cover, simply attach the Darwin Plates directly to the Carrier Trays, Skipping Steps 3-6.**